
Thompson Retractor

Anterior Cervical Set Up Tips + Tricks

Bilateral:

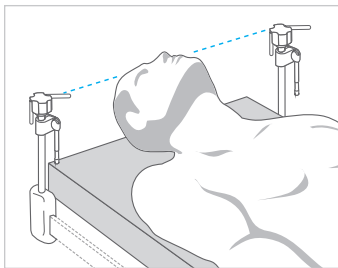
Rail Clamps

Placement

The rail clamps should be attached to the bed rail on either side of the patient, in line with the top of patient's head.

TIP: If no rail space is available near the patient's head, utilize a rail extender. (See below.)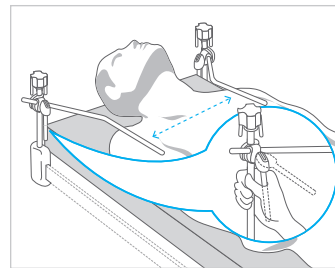


Angled Arms

Placement

Position arms "low and wide" creating a low profile working area and ensuring enough distance for handle retraction.

TIP: Keep arms level with the incision. Placing too low may interfere with imaging.

Unilateral: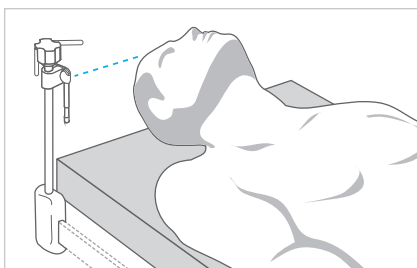

Rail Clamp


Placement

The rail clamp should be attached to the bed rail in line with the top of the patient's head.

TIP: If the rail clamp height is in the way, consider using the Bilateral Anterior Cervical System. (#SL91039)

TIP: The rail clamp may be positioned on either side of the table if obstructing the surgeon or assistant's movements.

TIP: If no rail space is available near the patient's head, utilize a rail extender. (See below.)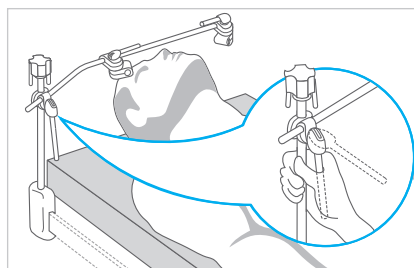


Crossbar

Placement

The crossbar should be positioned 2-3 fingers, level, above the patients head.

TIP: If more space is needed above the patient, the crossbar may be rotated upward.

TIP: When accessing uppermost cervical levels, an extension arm (#44014HS) can be added to the crossbar to help facilitate the appropriate angles of retraction. (See below.)

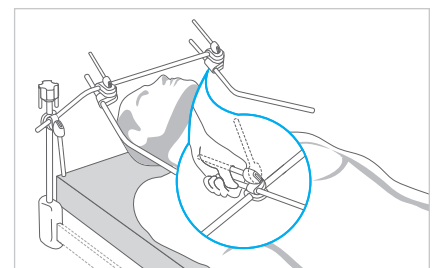Angled Arms

Placement

Position arms "low and wide" creating a low profile working area and ensuring enough distance for handle retraction.

TIP: Keep arms level with the incision. Placing too low may interfere with imaging.

TIP: When securing the arms to the crossbar, ensure the cam handle on the joint is in the same plane as the arms and not over the center of the crossbar taking up real-estate for the handles to be attached.

Frame Product Solutions / Add-Ons:


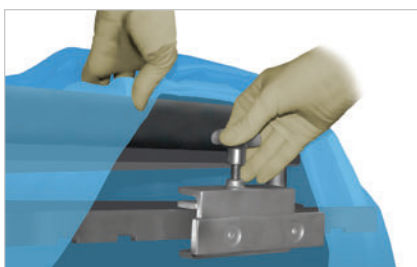
Table Adapter (#41927)

If using a Jackson spine table, the Jackson Frame Adapter may be connected to the table to add a standard bed rail to attach the Thompson frame.

Rail Extender (#5844)


If the OR table rail does not extend past the patient's head, a rail extender may be attached to the OR table rail to add length to the rail where needed. The rail clamp can then be secured to the rail extender.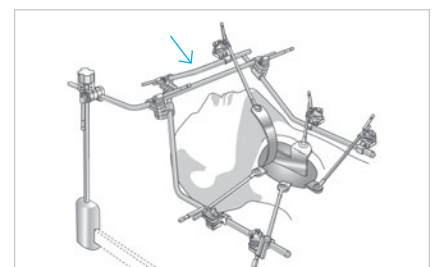

Extension Arm (#44014HS)

If desired, an extension arm may be added to the crossbar or angled arm to help facilitate additional upward angles of retraction.

Retractors: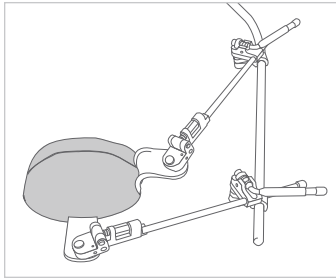


Handles

Placement

Handles can be clipped on to the arms or the crossbar at any location.

TIP: Use the S-Lock® feature to lock blades securely in place, while moving the retractor handle out of the way on the surgeon's access side.

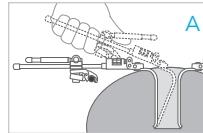TIPS: Utilizing S-Lock® technology, blades are easily placed by locking the blade in place on the handle and using in the same manner as a hand-held retractor to position (A). Once in place, the handle may be clipped onto the frame and secured (B). After clipping to the frame, blade angle may be further adjusted by using fingers or wrench to tow in or out (C).

NOTICE:

Release tension on retractors every 20 minutes to ensure proper blood flow.

Blades

Placement

Blades may be positioned in the head and toe direction as well as medially and laterally. Blades may be positioned for 1-4 levels of exposure from upper to lower cervical regions.

TIP: Most often, the need for toothed blades is eliminated through the use of a table mount.

NOTE: All anterior cervical blades shown below are radiolucent.

Concave

Medial / Lateral
35mm - 80mm

Concave Tapered

Medial / Lateral
35mm - 70mm

Micro

Microdiscectomy
35mm - 80mm

Longitudinal

Longitudinal
30mm - 70mm

Midline

Mandible
Left + Right

MIS Tapered

Medial / Lateral
35mm - 75mm

Odontoid

Odontoid /
Longitudinal Cephalad

Malleable

Longitudinal Cephalad
Multiple sizes available

Retractor Product Solutions / Add-Ons:


Manipulator Handle (#42228)

Can be used in addition to a Thompson Retractor Handle to make controlled retraction adjustments to the S-Lock blade position after the blade has been placed and attached to the frame.

Hand Held Adapter (#SL42128F)

Quickly and easily allows any Thompson S-Lock blade to become a hand held retractor. Ideal for when initial exposure requires hand held retractors or when retractors must be constantly repositioned.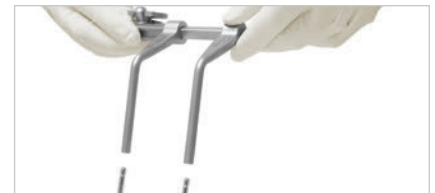

Cervical Distractors

Thompson Surgical now offers Left and Right Caspar Distractors, as well as distraction screws and screwdriver, for anterior cervical spine surgery.

Thompson
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

For a Free Trial Call **1-800-227-7543** *

* Free trial valid for U.S. customers only. Customers outside the U.S.A., please call +1-231-922-0177 for availability.

© 2017 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

© S-Lock®, PLA®, and the Thompson "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
Other patents pending.

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